

# TruCric User Manual



**An innovative and durable solution for practicing realistic front of neck access techniques**

**The TruCric model features a unique larynx insert facilitating a lifelike training experience for users practising needle/surgical cricothyroidotomy and percutaneous tracheostomy.**

**Easily interchangeable and cost-effective consumables make this model ideal for a busy teaching environment.**

**Contents**

**Product specifications .....2**

**Package contents.....2**

**Initial set-up information .....2**

**List of procedures facilitated by the TruCric.....2**

**Replacement of consumables ..... 3**

**Care and maintenance ..... 4**

**Warranty..... 4**

## Product specifications

Product Code: TCRIC1

Full shipment weight: Approx. 1kg

Full shipment dimensions: 41 x 26 x 23cm

## Package contents

- 1 TruCric ready-to-use (TCRIC1)
- 1 TruCric carrier case (CBAG03)
- 100ml bottle of lubrication (TL001)
- 10 Larynx inserts (NLX050 – 1 supplied on the manikin and 9 spare)
- 1 wrap around neck skin (RSN2005 – supplied on manikin)

## Initial set-up information

- Gently remove the model from the carrier case
- Secure the model to a suitable surface
- Please ensure the lung bag is attached to the airway tube
- Ensure the larynx is in correct position with the wraparound skin secured in place
- Spray generous amounts of lubrication provided with the model (TL001) on equipment prior to use

## List of procedures facilitated by the TruCric

- Needle and surgical cricothyroidotomy
- Percutaneous tracheostomy
- Air Jet ventilation

## Replacement of consumables

The larynx insert is designed to be replaced for each student to provide a unique training experience. This is a very quick and easy process, perfect for a busy teaching environment.

1. Remove the wraparound neck skin to provide access to the larynx insert
2. Remove the larynx insert and discard



3. Align a new insert into the larynx holder and press into place. Please ensure the ridges at all corners of the insert are tightly aligned with the larynx insert frame:



4. When replacing the neck skin ensure that it passes through the pre-defined gaps on the base, and the skin rests on the larynx insert to give prominent definition

## Care and maintenance

Store in clean, dry conditions away from heat and direct sunlight; avoid contact with metals, solvents, oils or greases and strong detergents. When the product is not in use please store in the black carrier case provided.

Thoroughly wash the AirSim X airway in warm water. Please use warm soapy water or something similar until all visible foreign matter and residue is removed.

Mild detergents or enzymatic cleaning agents may be used on the airway in accordance with the manufacturer's instructions and at the proper dilution. The detergent must not contain skin or mucous membrane irritants.

Please do not use any of the following when cleaning the AirSim product range

- Germicides, disinfectants, or chemical agents such as glutaraldehyde (e.g. Cidex®),
- Ethylene oxide, phenol-based cleaners or iodine-containing cleaners

**In response to the recent COVID-19 pandemic, we recommend this additional step to ensure the product is fully sanitised:**

Use alcohol spray (minimum 75%) and wipe off. Repeat this for 3-4 times to ensure to kill the virus completely.

## Warranty

TruCorp warrants this product to be free from defects in materials and workmanship, ensuring satisfactory performance for a period of one year from the date of delivery. This warranty provides customers with maximum coverage for each unit.

If a product malfunctions and is confirmed to be defective under warranty, TruCorp will repair or replace it at no cost. However, the warranty is void if the unit shows signs of tampering, excessive heat exposure, damage from sharp instruments, improper use, or any conditions beyond TruCorp's control. Consumable items such as wrap-around neck skins and larynx

inserts are not covered under warranty and will not be replaced. Damage resulting from misuse is not covered and may incur additional charges.

For full terms and conditions, visit: [TruCorp Warranty Policy](#)

**Please direct all warranty and repair inquiries to:**



**TruCorp Ltd**  
**33 Waringstown Road**  
**Lurgan, Co. Armagh**  
**BT66 7HH**  
**Northern Ireland**  
**[info@trucorp.com](mailto:info@trucorp.com)**  
**Tel: +44 (0) 28 3888 2714**